

tingletip

to help you experience or enhance orgasm



www.tingletip.com

clitoral stimulation

Many women have problems with sex at some stage in their life.

According to The Sexual Advice Association, sexual problems affect around 50% of women, and these can include loss of desire, loss of arousal or problems achieving orgasm.

Some women don't need to have an orgasm to enjoy sex, but inability to reach orgasm can be a problem for some women and their partners.

Psychosexual therapy can help a woman overcome orgasm problems. It involves exploring feelings about sex, herself and her relationship.

Many women find it difficult or impossible to reach orgasm through intercourse, but do orgasm with clitoral stimulation. This is perfectly normal. A recent survey found that over 60% of respondents never achieved orgasm from intercourse.

For many women, powerful massagers or vibrators such as **tingletip** can provide the level of clitoral stimulation needed to facilitate or enhance orgasm.

tingletip

tingletip was specifically designed to help women achieve or enhance orgasm.

tingletip utilises a unique and effective 'contra-rotating' action. This 'contra-rotating' design focuses the powerful vibrations to the tip and the action and frequency are optimised for female arousal.

The vibrational movement varies from low in the centre to being strongest at the edges, so by altering the angle, positioning and pressure of the tingletip head you can control both the intensity and focus of stimulation.

tingletip is powered by an electric toothbrush to provide a very discreet massager as it can be left charging ready to be used at any time without raising questions. It also means that you can take your massager on vacation without worrying about having to empty your suitcase at airport security.



actual size

using tingletip

Firstly ensure you are relaxed, comfortable and not likely to be interrupted or disturbed.

The electric toothbrush should be fully charged. To use, remove your electric toothbrush head and replace with tingletip massage head. Apply lubricant to the clitoral area before use. We recommend water based personal lubricants available from any pharmacy.

tingletip can be used alone, or with a partner. You may be more comfortable starting with your partner holding the body of the toothbrush but ideally you want to be in control so that you can vary the positioning and pressure to maximum effect.

Please note that **tingletip** is designed for external clitoral stimulation only and should not be inserted into the vagina.

After use, wash **tingletip** and your electric toothbrush thoroughly in warm, soapy water and dry.

your partner

Many men feel it is their role to make a woman orgasm through intercourse, and your partner may not understand or feel threatened by your need to use a massager to achieve orgasm.

If this is the case then you should talk together about this, research relevant websites or seek advice from a registered sex therapist.

Rather than being a threat to your partner, **tingletip** can enhance a relationship. The terms “experienced lover”, “knows what to do”, etc. all allude to a man who understands the importance of clitoral stimulation.

You may find that the best sex for both of you is achieved by you reaching orgasm by clitoral stimulation first and then following with intercourse. You will both be more relaxed and your partner can climax knowing that you have already achieved orgasm.



how to order

You can order **tingletip** from your therapist or from our website below.

A compatible electric toothbrush needs to be purchased separately to power **tingletip**. Compatible low-cost toothbrushes include Braun Oral-B Vitality Precision Clean and Sensitive Clean. See our website for full details and prices.

tingletip has been designed, manufactured and packaged to minimize environmental impact. Being powered by a rechargeable electric toothbrush has far less impact than using disposable batteries.

tingletip is made from medically safe, high-quality engineering plastics and is phthalate free.

Money-back guarantee. If you are not 100% satisfied with **tingletip** you can return for a full refund within 30 days of receipt.

www.tingletip.com

Creayta Ltd
PO Box 7352
READING
RG27 7EG
United Kingdom

phone: +44 (0)118 336 4480
email: info@creayta.com